

Checking Your Blood Glucose Using TRUE METRIX

A guide for testing your blood glucose

Simple Steps to Test Your Blood Glucose

Meter, Test Strips & System Components



Blood Glucose Testing




1

Wash your hands with warm, soapy water.



2

Remove Test Strip from vial and close vial immediately. Insert Test Strip into Test Port with Contact End (blocks facing up). The Meter turns on.

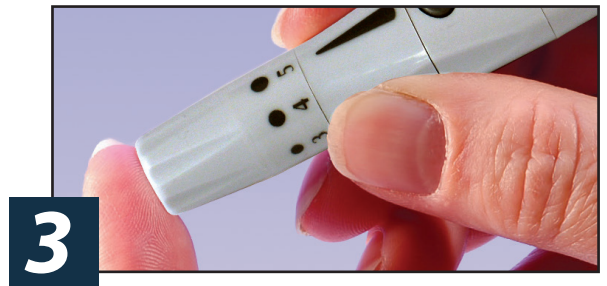

3

Lance your finger. For single patient use only.



4

With Test Strip still in Meter, touch Sample Tip to top of blood drop and allow blood to be drawn into Test Strip. Remove from blood drop immediately after Meter beeps and dashes appear across Meter Display.



5

In as fast as 4 seconds, glucose result will be displayed.



6

Hold meter with Test Strip pointing down. Press Strip Release Button to discard Test Strip in the appropriate container.

This guide provides a brief overview on testing your blood glucose. If you need information about setting up your meter with correct date and time, consult the "Getting Started" section of Owner's Booklet. For more detailed information on complete blood glucose testing procedure, consult "Testing Blood" section in the Owner's Booklet.

WARNING! Upon opening the test strip carton, examine the product for missing, damaged or broken parts. Ensure the test strip vial cap is securely closed. If the product is damaged or the vial cap is not closed, DO NOT use the test strips for testing; product may give inaccurate results. Contact Trividia Health Customer Care for replacement and assistance.

Distributed by:
Trividia Health UK Limited
27 Old Gloucester Street
London, United Kingdom
WC1N 3AX
UK: 0800 689 5035



EMERGO EUROPE
Westervoortsedijk 60,
6827 AT Arnhem,
The Netherlands

TRIVIDIA HEALTH, INC.
2400 N.W. 55TH COURT
FORT LAUDERDALE, FL 33309 U.S.A.

CE0197



GRAPHICS
Packaging

hamish 13aug20

INITIAL REQUESTOR DATE

Notes:

00-00-00 Notes:

ART INFORMATION

DESIGNER: Mark
DATE: 13aug20 SIZE: 8.5x11 4/0
SUBSTRATE/FORMAT:
White Board/Box Shrink Sleeve/Label
Pressure Sensitive/Label

IMAGES



Colors:

Graphic Artist/Reviewer:

<u>Artist/Date</u>	<u>Reviewer/Date</u>
mm13aug20	RW 10/02/20
RW 08/17/23	OR 08/17/23
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date
Artist/Date	Reviewer/Date

Process Colors Used in Art



4 Color Process

PANTONE® simulations used in this art may not match PANTONE-identified solid color standards. Use current PANTONE Color Reference Manuals for accurate color.